


FH-1001 Reusable Forehead Thermometer

C-8702R Thermometer

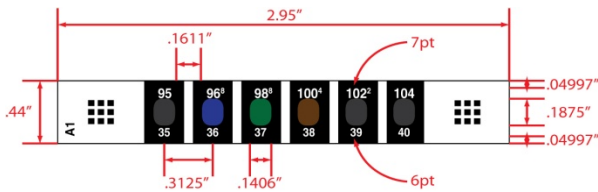

Thermometer	C-8702R
Instruction	Standard Blue no name
Instruction size	3.625" x 1.875" ± 0.031"
Instruction Paper	Qualified 70# coated paper
Art File Number	FRHDInst-002-Blue Forehead Instruction Folder



***To meet the demands of the CoronaVirus pandemic, the FH-1001 is only being offered in Bulk Orders with the Sleeves included separately in the shipment. Artwork may vary on sleeve.**



Forehead Thermometer Instructions
1. How to use: Hold thermometer at both ends, press firmly on dry forehead hold until colors stop changing (about 15 seconds). Read thermometer while still on forehead.
2. Read temperature as follows: Green = correct temperature. If you do not see green, then look for the blue and tan colors. Blue add 1°F, Tan subtract 1°F.
3. Thermometer should be used indoors at room temperature approximately 68°-86°F. Avoid direct sunlight or strong lamps when taking temperature.
4. Do not take temperature if individual has been eating, drinking or exercising within the last 30 minutes or just coming in from the outdoors.
5. Always consult your physician if a fever exists.
6. May be cleaned with a soft cloth and should be stored inside instruction sheet.
Forehead skin temperatures are lower than oral temperatures. The readings shown on this thermometer have been adjusted 4.5°F to approximate the difference between forehead and oral temperatures during normal to feverish conditions. Difference may be from 3.8°F to 6°F. A clinical thermometer may be used if oral temperature differs from the numbers shown.
KEEP OUT OF REACH OF CHILDREN, TO BE ADMINISTERED BY ADULTS ONLY



PACKAGING/SHIPPING

Ship per Customer request

Tempagenix, LLC is the WBENC woman-owned certified distributor of FH-1001. The product is manufactured in an ISO-certified facility by American Thermal Instruments in Dayton, OH.